HOMEDICS®

MiBody 360 Blood Pressure Monitor

Instruction Manual

360BP-EU

3 year
guarantee

Fig.1

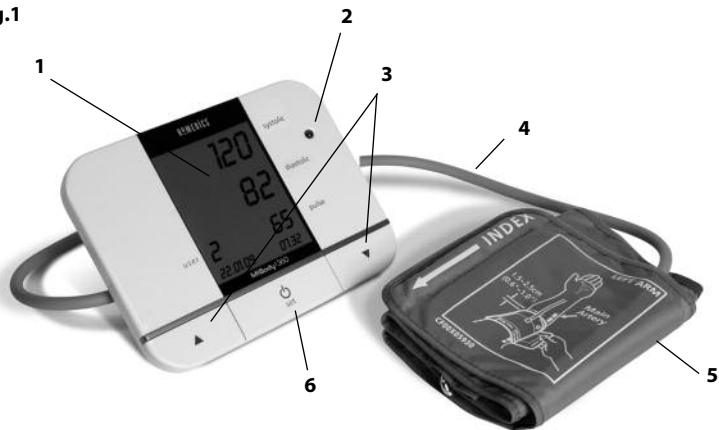
Fig.2

Fig.3

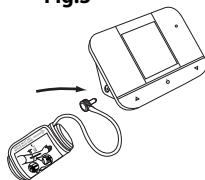
Fig.4

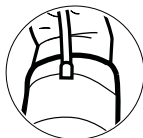
Fig.5

Fig.6

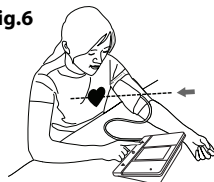
Fig.7

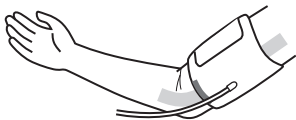
Fig.8


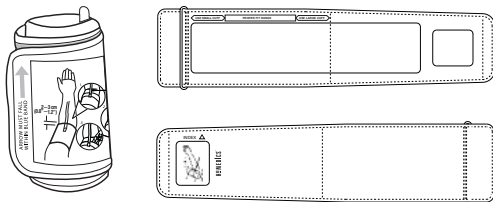
Fig.9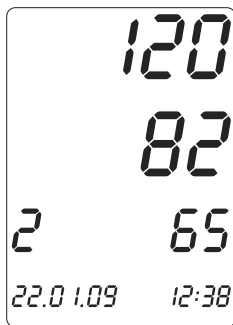

Quick Start Guide

TO DOWNLOAD FULL INSTRUCTION MANUAL AND INSTALL MIBODY 360 TRACKING SOFTWARE PLEASE VISIT:

www.mibody360.com

TO START USING YOUR HOMEDICS MIBODY360 BLOOD PRESSURE MONITOR STRAIGHT AWAY SIMPLY FOLLOW THE QUICK START INSTRUCTIONS BELOW:

Before use

Before the blood pressure monitor is taken into use a number of basic steps should be observed. Please read this manual thoroughly before the blood pressure monitor is used for the first time and keep the manual for future use.

The blood pressure monitor is intended for home use only. Any measurement is solely for information and can under no circumstances replace a visit to the doctor. The blood pressure measurements should always be assessed by a qualified person (a doctor, nurse or similar) who is familiar with the user's general medical condition. By regular use and by keeping a record of the results, this person can be kept well informed about developments in the user's blood pressure. Measurements must never be interpreted by the user with the aim of changing levels in the use of medicine as prescribed by a doctor, but the doctor's instructions must be followed **AT ALL TIMES**.

Use of appropriate cuff size is crucial to correct measurement. Follow the instructions presented in this manual and printed on the cuff to ensure that the correct cuff size is used.

The blood pressure monitor is not suited for users who suffer from arrhythmia (heart rhythm disturbances) and errors may occur in the measurement results if the user has suffered a stroke, suffers from cardiovascular diseases, has a very low blood pressure, or suffers from other symptoms such as circulatory diseases (diabetes, kidney diseases, arterial sclerosis (deposits in the arteries), or poor peripheral blood circulation (e.g. in hands and feet).

Electromagnetic disturbances: The device contains sensitive electronic components. Keep it clear of strong electric or electromagnetic fields in the immediate surroundings (e.g. mobile telephones, microwave ovens) as these may temporarily reduce measurement accuracy.

Use the blood pressure monitor only for its original purpose.

The device is intended for measurement of blood pressure and pulse with adult persons. Do not apply the monitor to infants nor to persons who cannot express their permission. The device must not be operated by children.

The blood pressure monitor functions in accordance with the oscillometric measurement principle. These blood pressure changes are measured in the cuff on the basis of the arm pulse and are converted into a numerical value of the current blood pressure. Simultaneously, the number of pulse beats are registered and calculated as number of pulse beats per minute.

There are two kinds of discernable blood pressures. Both are expressed in mmHg (millimeters on the mercury column): The systolic and the diastolic. The systolic ('the high blood pressure') states the pressure when the heart chambers contract to send blood through your body, and the diastolic ('the low blood pressure') states the pressure when the heart is at ease and filling with blood before the next muscle contraction. The connection between values (mmHg) is expressed as e.g. 130 over 85 which means that the systolic pressure is 130 and the diastolic pressure is 85.

Why is it important to keep an eye on your blood pressure?

Many of today's most common diseases are connected to hypertension, also referred to as 'high blood pressure'. Hypertension is closely connected to cardiovascular diseases, and for people in the danger zone the blood pressure provides an important tool to keep an eye on the development.

Standard for blood pressure

The WHO blood pressure standard defines the areas for high and low blood pressure. The standard is for guidance only as individual blood pressure levels vary from person to person, across age groups, etc.

LED light indicator	World Health Organisation Classification (WHO)	Systolic pressure (mmHg)	Diastolic pressure (mmHg)
GREEN	Optimal	<120	<80
	Normal	120-129	80-84
	High Normal	130-139	85-89
AMBER	Hypertension grade 1	140-159	90-99
RED	Hypertension grade 2	160-179	100-109
	Hypertension grade 3	≥180	≥110

How to use the blood pressure monitor

This MiBody 360 blood pressure monitor is equipped with smart technology. By pressing a single button the cuff is automatically inflated to block the blood pressure flow through the artery. Subsequently, the deflation process starts. The smart technology enables automatic inflation and deflation to be adjusted exactly to fit each individual. Please be aware that any motion during inflation or deflation will cause measurement errors. When the measurement is complete the display will show the systolic pressure, the diastolic pressure, and the heart rate.

MiBody 360 blood pressure monitor is approved to provide clinically correct results.

The device is approved by and in accordance with the demands of MDD (Medical Device Directive, European Norm EN-1060-3 &/or 1060-4) as well as the guidelines from The British Hypertension Society (BHS) accreditation.

Description of the parts (referring to Fig.1 page 2)

1. LCD display
2. LED indicator bulb
3. User select buttons x 2
4. Air tube and connector
5. Arm cuff
6. ON/OFF button + SET

Inserting and changing batteries

1. Press slightly on the handle of door while you lift and remove it. Insert or change the batteries in the battery compartment. Make sure that the batteries are placed correctly. The door is put back in its place.
2. The batteries must be changed when 'batt lo' appears on the display.

As the supplied batteries are for test only, they may be discharged earlier than batteries you buy in stores. Replace the batteries in pairs. Remove batteries when unit is not in use for extended periods of time.

How to use the blood pressure device

First time use and setting date/time:

1. First time use after inserting the batteries 'SET UP' will be displayed
2. Press 'set' display will show 'dAtE':
3. Day will flash, use the up or down arrows to chose date, press 'set' to confirm and move to month, follow the procedure for month and move to year select and press 'set'
4. Display will show 'hour': Hour will flash, use the up or down arrows to chose hour, press 'set' to confirm and move to minutes, follow the produce for hour and press 'set'
5. The blood pressure monitor is now set-up and ready to use, display will show '0' '0' '0' plus date, time & user number.
6. If you need to change date/time during normal use press and hold 'set' to enter 'SET UP' and repeat procedure from step 1.

Note: after 30 seconds the display will revert to default start screen '0' '0' '0'

Fitting the cuff

1. If the cuff is not assembled at first use, put the end of the cuff which is furthest away from the tube through the D-ring to form a noose. (Fig.2)
2. Apply the tube to the left side of the device. (Fig.3)
3. Remove tight-fitting clothing from your left upper arm and position the cuff at approximately 2-3 cm above the elbow so that the tube is centered at the middle of the arm. The tube must face downward towards the hand. (Fig.4)
4. Pull gently at the end of the cuff so that it tightens easily and smoothly around the arm. The cuff is secured by pulling the velcro fastener towards the cuff.
5. You should be able to fit two fingers between the cuff and your arm. The cuff should not be fitted any tighter than this. (Fig.5)
6. Relax your body and rest your arm on a table with your palm up so that the cuff is positioned at the same height as the heart. Make sure that the tube does not bend. (Fig.6)

If it is not possible to fit the cuff on the left arm, it can be placed on the right arm instead. However, it is important that all measurements are carried out on the same arm. If the measurement is to be taken on the right arm, the tube on the cuff should be positioned above the main artery. Press the arm's arteries with two fingers at approximately 2 cm above the elbow on the inside of the arm to find out where the pulse is strongest.

This is the main artery. (Fig.7)

IMPORTANT!

The size of the cuff is instrumental in ensuring accurate measurement results. The cuff has an appropriate size if the INDEX arrow is placed within the two OK arrows as illustrated below. If the arrow falls outside this area, another cuff (XL or XS) must be used. (Fig.8)

Important

- The blood pressure changes with every heartbeat and is consequently changing constantly 24 hours a day.
- The measurement of blood pressure can be influenced by the user's position, condition, or other factors. To obtain the highest degree of accuracy it is recommended that the measurement is taken 1 hour after workout, bathing, finishing a meal, consuming alcoholic or caffeine beverages, or after smoking.
- Before measurement it is recommended that you sit still and relax for 15 minutes. The user should not be physically tired or exhausted when the measurement is taken.
- The user must not talk or move arm or hand during measurement.

How to measure the blood pressure

1. Press 'set' to turn the unit on
2. Select user (1-6 or 0 for guest) using up/down arrows
3. Ensure cuff is correctly positioned on arm, ensuring you are sitting comfortably with the cuff at the same level as your heart.
4. Press 'set' to start blood pressure measurement - the cuff will inflate and start measurement
5. When the measurement is complete the cuff will automatically deflate and the display will show 'diastolic' and 'systolic' pressure readings as well as 'pulse', the LED indicator bulb will light to give a health status indication based on the WHO standard (see WHO table on page 5). The measurement will be simultaneously stored in the device memory (only users 1-6). (Fig.9)
6. The device will auto off after 30 seconds or when pressing 'set'
7. Another measurement can be taken straight away if necessary simply return to step 1 and repeat all steps.

Error messages:

	An Error has been detected during measurement. Press 'set' to stop and start measurement again. If the problem persists please contact HoMedics customer service as indicated in the guarantee leaflet
---	--

Worth knowing about blood pressure measurement

- Blood pressure measurements should be performed in quiet and comfortable surroundings as measurements can be affected by high or low temperatures. The blood pressure should be measured at normal body temperatures.
- Do not move or speak during measurement. Keep your feet flat on the floor.
- Do not touch the cuff or the blood pressure device during measurement.
- To obtain the best possible measurement, the blood pressure should be measured at approximately the same time every day, and on the same arm.
- If you are to perform several successive measurements, please wait 5 minutes between each measurement.
- The measurements are for guidance only. If you have doubts about your blood pressure, consult your doctor.
- The product is not suited for persons suffering from arrhythmia (heart rhythm disturbances).
- Measurement errors may occur if the user suffers from diabetes, poor blood circulation, kidney problems, or has suffered a stroke.

Memory

This blood pressure device can be used by 6 users. Each user can store up to 30 measurements before download.

When user memory is full the display will show FULL and user number 1, 2, 3 The Amber LED will light.

Please follow the full manual instructions to download to MiBody 360 software.

Error search

If irregularities occur during use, please check the following items before you contact your vendor:

Symptom	Possible reasons	Correction
The device does not switch on when the stop/start button is activated.	A change of batteries is required.	Change to four new batteries.
Erro measurement error symbol in the display, or the blood pressure measurement is excessively low (or high).	The cuff is fitted incorrectly.	Fit the cuff again so that it is positioned correctly.
	Did you talk or move during measurement?	
	Shaking of the arm while wearing the cuff.	Check the cuff's connection. Restart measurement.
	Airflow is irregular. The tube is not properly applied to the device.	

If the device still does not function, please contact the store where the device was purchased or contact HoMedics customer service details located on guarantee leaflet.

Do not attempt to open the device or adjust any of the inner parts yourself.

Specifications

Power source	4 AA batteries
Measurement standard	Oscillometric
Measurement zone	Pressure: 30-280 mmHg Pulse: 40-200 beats per minute
Accuracy	Pressure: ± 3 mmHg Pulse: Reading inside $\pm 5\%$
Pressure sensor	Semi conductor
Inflation	Automatic - pump-driven
Deflation	Automatic - overpressure valve
Memory capacity	30 measurements per user (180 total)
Auto off	30 seconds after latest use
Conditions during use	10°C - 40°C 85% RH max
Storage conditions	-20°C - 60°C 85% RH max
Cuff circumference	22-32 cm
Age limitation	18 years
Classification	 BF type. Device and cuff are designed to provide special protection against electric shock.

Specifications can be changed without notice.

Cleaning and maintenance

1. The blood pressure monitor, the cuff, and the storage box can be rinsed with a soft cloth moistened with a mild detergent and wiped dry with a soft, dry cloth or paper towel. Never use strong, caustic detergents as these may cause damage to the synthetic parts. Also, never use diluents, alcohol or kerosene to clean the device.

Safety and electrical devices

1. When the device is not used, store it in the storage box.
2. Only use the device on wrist or upper arm, not on other body parts.
3. When the device is not used for extended periods of time, it is recommended that you remove the batteries to avoid leakage which may cause damage to the device.
4. If the device has been stored at very low temperatures near the freezing point, please allow it to obtain room temperature before using it again.
5. Neither device nor cuff must be disassembled as this may cause serious damage to the device. If repair is necessary, the device must be returned to the store where it was purchased. Do not attempt to open the device or adjust the inner parts yourself.

6. Do not submerge the device in water or other liquids as this will cause damage to the device.
7. Do not expose the device nor the cuff to extreme temperatures, moisture or direct sunlight. Protect the device against dust.
8. The cuff and the tube should not be folded up tightly. The device must not be activated if it is not appropriately fitted to wrist or upper arm.
9. Avoid dropping the device and do not subject it to heavy pressure.
10. The device must not be exposed to extreme shocks such as being dropped on the floor.
11. The device is intended for use in private households only and is only suited for use with adults (over 18 years of age).
12. If the highly unlikely incident should occur that the cuff continues to inflate and does not stop, immediately remove the cuff from wrist or upper arm.
13. The device must be stored at temperatures indicated on specification to ensure accurate measurements.

Protecting the environment and disposing of batteries

Respect the environment when you change batteries. Spent batteries are not ordinary household waste. They should be disposed of at authorized collection points. Make sure that the batteries are not too old or worn out as there is a risk that they might leak acid. Check the expiry date on the batteries. Remember to remove the batteries before disposing of the product.

■ This marking indicates that this product should not be disposed with other household wastes throughout the EU. To prevent possible harm to the environment or human health from uncontrolled waste disposal, recycle it responsibly to promote the sustainable reuse of material resources. To return your used device, please use the return and collection systems or contact the retailer where the product was purchased. They can take this product for environmental safe recycling. (WEEE explanation)

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